INMIND

CLEANER SERVICE Co.,Ltd.

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Tél : +32 (0)67 89 41 00 • Fax : +32 (0)67 84 37 67
www.huckerts.net

See our general conditions of sale

UMONIUM³⁸® efficiency against MERS-CoV?

Dr Nicolas COLTEL

head of Scientific department of Laboratoire Huckert's International

Background

Coronaviruses belong to a family of viruses that can infect a variety of mammalian hosts. Coronaviruses are readily transmissible among humans by direct contact with respiratory secretions, body fluids and excretions from infected individuals (1). In humans, coronaviruses are mainly responsible for the common cold; however, the severe acute respiratory syndrome coronavirus (SARS-CoV) can cause severe respiratory illness (outbreaks in 2002-2003, (2)). Middle East Respiratory Syndrome Coronavirus (MERS-CoV) was then discovered in Saudi Arabia, Qatar, and Jordan and, as of September 13, 2013, has caused severe respiratory illness. More recently human transmission has just been described in Germany, Korea, Thailand and China. Based on the clinical presentation of severe respiratory illness and the low risk to public health due to limited human-to-human transmission, this novel coronavirus is classified as a Risk Group 3 (RG3) human pathogen requiring Containment Level 3).

Evidences

According to the Canadian Health Authority (3), MERS-CoV could be inactivated by common disinfection measures such as a 5 minute contact of household bleach (4), ice-cold acetone, ice-cold acetone/methanol mixture (40:60), 70% ethanol (10 minutes), 100% ethanol (5 minutes) paraformaldehyde, and glutaraldehyde (5). MERS-CoV is also sensitive to heat (60°C for 30 minutes) (5-6), and UV radiation (7).

Sattar et al (8) demonstrated that classical coronaviruses are susceptible to 0.1% sodium hypochlorite, 0.1% organochlorine, 10% iodophore, 70% ethanol and 2% glutaraldehyde, but resistant to 0.04% quaternary ammonium compound and phenolics.

WHO (2), CDC (1) and European different authorities recommend health and hygiene precautions to be reinforced. Special considerations have to taken to prevent air and contact transmission.

In France and Belgium, practical recommendations edited in 2013 during the first SARS outbreak are still applicable (9). To date, no disinfectant has been tested on MERS-CoV. EN14476:2013 is based on Poliovirus, Adenovirus and Norovirus. Poliovirus is known to be particularly resistant. So the precaution strategy encourages to use disinfectants that demonstrate a virucidal activity on Poliovirus according to EN14476 (9).

Conclusion

Umonium38® is broad spectrum disinfectant, and is effective against naked viruses according to EN14476:2013. Umonium38® products demonstrated their efficiency upon SARS virus during the previous outbreak. So we could think that Umonium38® is efficient on Middle East Respiratory Syndrome Coronavirus (MERS-CoV). To ensure a potential efficacy, give advantage to ready to use sprays (Umonium38® Medical Spray) or at least 2.5% dilutions (from concentrated formulas) with a minimum contact time of 10-15 minutes.

Recommendations

The use of Umonium38® products will provide safety for the materials (see our procedure), but it does not exempt users to wear protective elements (gloves, glasses, apron, long-sleeved gown...etc) to protect themselves from any virus spillage. Every waste material used for the disinfection (tissues, apron, gloves, long-sleeved gown....) should be destructed as biological/infectious material.

Personal Protective elements must also be cleaned and disinfected prior to their reuse.

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UMONIUM38 INSTRUMENTS

Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

Date of issue: 7/16/2015

Revision date: 2/22/2018

Supersedes: 3/9/2017 Version: 3.4

SECTION 1: Identification of the substance/mixture and of the company/undertaking

1.1. Product identifier

Product form : Mixture
Product name : UMONIUM38 INSTRUMENTS

1.2. Relevant identified uses of the substance or mixture and uses advised against

1.2.1. Relevant identified uses

Main use category : Industrial use, Professional use, Consumer use
Industrial/Professional use spec : Hospital.
Pharmaceutical industry.
Professional uses: Public domain (administration, education, entertainment, services, craftsmen)
Industrial
Use of the substance/mixture : Public sector
Health services
Professional uses: Public domain (administration, education, entertainment, services, craftsmen)
Function or use category : Disinfectant, Medical devices

1.2.2. Uses advised against

No additional information available

1.3. Details of the supplier of the safety data sheet

Fabricant / Notifiant / Editeur FDS:

Laboratoire HUCKERT'S INTERNATIONAL

Avenue Lavoisier, 20-22

B-1300 WAVRE

+32 (0) 67 89 41 00

+32 (0) 67 84 37 67

info@huckerts.net

1.4. Emergency telephone number

Emergency number : +32 (0)70-245 245

SECTION 2: Hazards identification

2.1. Classification of the substance or mixture

Classification according to Regulation (EC) No. 1272/2008 [CLP]

Serious eye damage/eye irritation, Category H319

2

Full text of H statements : see section 16

Adverse physicochemical, human health and environmental effects

No additional information available

2.2. Label elements

Labelling according to Regulation (EC) No. 1272/2008 [CLP]

Hazard pictograms (CLP)

GHS07

Signal word (CLP) : Warning
Hazard statements (CLP) : H319 - Causes serious eye irritation.
Precautionary statements (CLP) : P305+P351+P338 - IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing.
P332+P313 - If skin irritation occurs: Get medical advice/attention.
P337+P313 - If eye irritation persists: Get medical advice/attention.

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UMONIUM38 INSTRUMENTS Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

2.3. Other hazards

No additional information available

SECTION 3: Composition/information on ingredients

3.1. Substances

Not applicable

3.2. Mixtures

Name	Product identifier	%	Classification according to Directive 67/548/EEC	Classification according to Regulation (EC) No. 1272/2008 [CLP]
Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16))	(CAS-No.) 68424-85-1 (EC-No.) 270-325-2	1 - 10	Xn; R22 C; R34 N; R50	Acute Tox. 4 (Oral), H302 Skin Corr. 1B, H314 Aquatic Acute 1, H400
Parfum		1-2.0	Xn; R22 R43 N; R51/53 R10	Flam. Liq. 3, H226 Skin Sens. 1, H317 Asp. Tox. 1, H304 Aquatic Chronic 2, H411
Parfum		0.1-1.0	Xn; R38 R43 N; R51/53 R10	Flam. Liq. 3, H226 Skin Sens. 1, H317 Asp. Tox. 1, H304 Aquatic Chronic 1, H410

Full text of R- and H-statements: see section 16

SECTION 4: First aid measures

4.1. Description of first aid measures

- First-aid measures general : Never give anything by mouth to an unconscious person. If you feel unwell, seek medical advice (show the label where possible).
- First-aid measures after inhalation : Assure fresh air breathing. Allow the victim to rest.
- First-aid measures after skin contact : Remove affected clothing and wash all exposed skin area with mild soap and water, followed by warm water rinse.
- First-aid measures after eye contact : Rinse immediately with plenty of water. Obtain medical attention if pain, blinking or redness persists.
- First-aid measures after ingestion : Rinse mouth. Do NOT induce vomiting. Obtain emergency medical attention.

4.2. Most important symptoms and effects, both acute and delayed

- Symptoms/effects : Not expected to present a significant hazard under anticipated conditions of normal use.

4.3. Indication of any immediate medical attention and special treatment needed

No additional information available

SECTION 5: Firefighting measures

5.1. Extinguishing media

- Suitable extinguishing media : Foam. Dry powder. Carbon dioxide. Water spray. Sand.
- Unsuitable extinguishing media : Do not use a heavy water stream.

5.2. Special hazards arising from the substance or mixture

No additional information available

5.3. Advice for firefighters

- Firefighting instructions : Use water spray or fog for cooling exposed containers. Exercise caution when fighting any chemical fire. Prevent fire-fighting water from entering environment.
- Protection during firefighting : Do not enter fire area without proper protective equipment, including respiratory protection.

SECTION 6: Accidental release measures

6.1. Personal precautions, protective equipment and emergency procedures

6.1.1. For non-emergency personnel

- Emergency procedures : Evacuate unnecessary personnel.

6.1.2. For emergency responders

- Protective equipment : Equip cleanup crew with proper protection.
- Emergency procedures : Ventilate area.

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UMONIUM38 INSTRUMENTS Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

6.2. Environmental precautions

Do not allow product to spread into the environment. Avoid release to the environment. Prevent soil and water pollution.

6.3. Methods and material for containment and cleaning up

Methods for cleaning up : Soak up spills with inert solids, such as clay or diatomaceous earth as soon as possible. Collect spillage. Store away from other materials.

6.4. Reference to other sections

See Heading 8. Exposure controls and personal protection.

SECTION 7: Handling and storage

7.1. Precautions for safe handling

Precautions for safe handling : Wash hands and other exposed areas with mild soap and water before eating, drinking or smoking and when leaving work. Provide good ventilation in process area to prevent formation of vapour.

7.2. Conditions for safe storage, including any incompatibilities

Storage conditions : Keep only in the original container in a cool, well ventilated place away from : Keep away from sources of ignition. Keep container closed when not in use.

Incompatible products

: Strong bases. Strong acids.

Incompatible materials

: Sources of ignition. Direct sunlight.

7.3. Specific end use(s)

No additional information available

SECTION 8: Exposure controls/personal protection

8.1. Control parameters

UMONIUM38 INSTRUMENTS & EQUIPMENTS		
Belgium	Local name	
Belgium	Limit value (mg/m ³)	500 mg/m ³
Belgium	Limit value (ppm)	200 ppm
Belgium	Short time value (mg/m ³)	1000 mg/m ³
Belgium	Short time value (ppm)	400 ppm

8.2. Exposure controls

Personal protective equipment:

Avoid all unnecessary exposure.

Hand protection:

In case of repeated or prolonged exposure : In case of repeated or prolonged exposure : Wear protective gloves.

Eye protection:

Chemical goggles or safety glasses

Respiratory protection:

No special protection required where adequate ventilation is maintained

Other information:

Do not eat, drink or smoke during use.

SECTION 9: Physical and chemical properties

9.1. Information on basic physical and chemical properties

Physical state	: Liquid
Colour	: Green.
Odour	: characteristic.
Odour threshold	: No data available
pH	: 7
pH solution	: 7
Relative evaporation rate (butylacetate=1)	: No data available
Melting point	: No data available
Freezing point	: No data available
Boiling point	: 93 °C
Flash point	: 35 °C

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UMONIUM38 INSTRUMENTS Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

Auto-ignition temperature	: No data available
Decomposition temperature	: No data available
Flammability (solid, gas)	: Non flammable.
Vapour pressure	: < 5 hPa
Relative vapour density at 20 °C	: No data available
Relative density	: No data available
Solubility	: Soluble in water. Water: 100 % Ethanol: 100 %
Log Pow	: 0.48
Viscosity, kinematic	: No data available
Viscosity, dynamic	: No data available
Explosive properties	: No data available
Oxidising properties	: No data available
Explosive limits	: No data available

9.2. Other information

No additional information available

SECTION 10: Stability and reactivity

10.1. Reactivity

No additional information available

10.2. Chemical stability

Not established.

10.3. Possibility of hazardous reactions

Not established.

10.4. Conditions to avoid

Direct sunlight. Extremely high or low temperatures.

10.5. Incompatible materials

Strong acids. Strong bases.

10.6. Hazardous decomposition products

fume. Carbon monoxide. Carbon dioxide.

SECTION 11: Toxicological information

11.1. Information on toxicological effects

Acute toxicity (oral)	: Not classified
Acute toxicity (dermal)	: Not classified
Acute toxicity (inhalation)	: Not classified

UMONIUM38 INSTRUMENTS

LD50 oral rat	5.84 ml/kg
---------------	------------

Perfume

LD50 oral rat	<= 2480 mg/kg
LD50 dermal rabbit	<= 2480 mg/kg

Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16) (68424-85-1)

LD50 oral rat	≈ 344 mg/kg
LD50 dermal rabbit	≈ 3340 mg/kg

Perfume

LD50 oral rat	> 5 g/kg
LD50 oral	<= 4.8 g/kg

Skin corrosion/irritation	: Causes skin irritation. pH: 7
---------------------------	------------------------------------

Additional information	: Based on available data, the classification criteria are not met
Serious eye damage/irritation	: Causes serious eye irritation. pH: 7

Additional information	: Based on available data, the classification criteria are not met
Respiratory or skin sensitisation	: Not classified
Additional information	: Based on available data, the classification criteria are not met

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UMONIUM38 INSTRUMENTS Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

Germ cell mutagenicity	: Not classified
Additional information	: Based on available data, the classification criteria are not met
Carcinogenicity	: Not classified
Additional information	: Based on available data, the classification criteria are not met
Reproductive toxicity	: Not classified
Additional information	: Based on available data, the classification criteria are not met

STOT-single exposure	: Not classified
Additional information	: Based on available data, the classification criteria are not met
STOT-repeated exposure	: Not classified
Additional information	: Based on available data, the classification criteria are not met
Aspiration hazard	: Not classified
Additional information	: Based on available data, the classification criteria are not met

Potential adverse human health effects and symptoms	: Based on available data, the classification criteria are not met
---	--

SECTION 12: Ecological information

12.1. Toxicity

Acute aquatic toxicity	: Not classified.
Chronic aquatic toxicity	: Not classified.

UMONIUM38 INSTRUMENTS

LC50 fish 1	13.8 mg/l
LC50 other aquatic organisms 1	0.55 mg/l
EC50 Daphnia 1	0.39 mg/l

Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16) (68424-85-1))

LC50 fish 1	0.28 mg/l
NOEC (acute)	<
TLM fish 1	<

12.2. Persistence and degradability

UMONIUM38 INSTRUMENTS

Persistence and degradability	Not established.
Biochemical oxygen demand (BOD)	19.5 g O ₂ /g substance
Chemical oxygen demand (COD)	< 5 g O ₂ /g substance
Biodegradation	81.1 %

Perfume

Persistence and degradability	May cause long-term adverse effects in the environment.
-------------------------------	---

12.3. Bioaccumulative potential

UMONIUM38 INSTRUMENTS

Log Pow	0.48
Bioaccumulative potential	Not established.

Perfume

Bioaccumulative potential	Not established.
---------------------------	------------------

12.4. Mobility in soil

No additional information available

12.5. Results of PBT and vPvB assessment

Component	
Alkyl (C12-16) dimethylbenzyl ammonium chloride (ADBAC/BKC (C12-16) (68424-85-1))	This substance/mixture does not meet the PBT criteria of REACH regulation, annex XIII This substance/mixture does not meet the vPvB criteria of REACH regulation, annex XIII

12.6. Other adverse effects

Additional information	: Avoid release to the environment.
------------------------	-------------------------------------

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UMONIUM38 INSTRUMENTS

Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

SECTION 13: Disposal considerations

13.1. Waste treatment methods

- Product/Packaging disposal recommendations : Dispose in a safe manner in accordance with local/national regulations.
Ecology - waste materials : Avoid release to the environment.

SECTION 14: Transport information

In accordance with ADR / RID / IMDG / IATA / ADN

14.1. UN number

- UN-No. (ADR) : Not applicable
UN-No. (IMDG) : Not applicable
UN-No. (IATA) : Not applicable
UN-No. (ADN) : Not applicable
UN-No. (RID) : Not applicable

14.2. UN proper shipping name

- Proper Shipping Name (ADR) : Not applicable
Proper Shipping Name (IMDG) : Not applicable
Proper Shipping Name (IATA) : Not applicable
Proper Shipping Name (ADN) : Not applicable
Proper Shipping Name (RID) : Not applicable

14.3. Transport hazard class(es)

ADR

- Transport hazard class(es) (ADR) : Not applicable

IMDG

- Transport hazard class(es) (IMDG) : Not applicable

IATA

- Transport hazard class(es) (IATA) : Not applicable

ADN

- Transport hazard class(es) (ADN) : Not applicable

RID

- Transport hazard class(es) (RID) : Not applicable

14.4. Packing group

- Packing group (ADR) : Not applicable
Packing group (IMDG) : Not applicable
Packing group (IATA) : Not applicable
Packing group (ADN) : Not applicable
Packing group (RID) : Not applicable

14.5. Environmental hazards

- Dangerous for the environment : No
Marine pollutant : No
Other information : Hazardous to the aquatic environment — Chronic Hazard, Category 4

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UMONIUM38 INSTRUMENTS

Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

- Rail transport

No data available

14.7. Transport in bulk according to Annex II of Marpol and the IBC Code

Not applicable

SECTION 15: Regulatory information

15.1. Safety, health and environmental regulations/legislation specific for the substance or mixture

15.1.1. EU-Regulations

Contains no REACH substances with Annex XVII restrictions

Contains no substance on the REACH candidate list

Contains no REACH Annex XIV substances

15.1.2. National regulations

No additional information available

15.2. Chemical safety assessment

No chemical safety assessment has been carried out

SECTION 16: Other information

Indication of changes:

Revision - See : *

Data sources

: REGULATION (EC) No 1272/2008 OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL of 16 December 2008 on classification, labelling and packaging of substances and mixtures, amending and repealing Directives 67/548/EEC and 1999/45/EC, and amending Regulation (EC) No 1907/2006.

Training advice

: Normal use of this product shall imply use in accordance with the instructions on the packaging.

Other information

: None.

Full text of R-, H- and EUH-statements:

Acute Tox. 4 (Dermal)	Acute toxicity (dermal), Category 4
Acute Tox. 4 (Oral)	Acute toxicity (oral), Category 4
Aquatic Acute 1	Hazardous to the aquatic environment — Acute Hazard, Category 1
Aquatic Chronic 1	Hazardous to the aquatic environment — Chronic Hazard, Category 1
Aquatic Chronic 2	Hazardous to the aquatic environment — Chronic Hazard, Category 2
Asp. Tox. 1	Aspiration hazard, Category 1
Eye Irrit. 2	Serious eye damage/eye irritation, Category 2
Flam. Liq. 2	Flammable liquids, Category 2
Flam. Liq. 3	Flammable liquids, Category 3
Skin Corr. 1B	Skin corrosion/irritation, Category 1B
Skin Irrit. 2	Skin corrosion/irritation, Category 2
Skin Sens. 1	Skin sensitisation, Category 1
STOT SE 3	Specific target organ toxicity — Single exposure, Category 3, Narcosis
STOT SE 3	Specific target organ toxicity — Single exposure, Category 3, Respiratory tract irritation
H225	Highly flammable liquid and vapour.
H226	Flammable liquid and vapour.
H302	Harmful if swallowed.
H304	May be fatal if swallowed and enters airways.
H312	Harmful in contact with skin.
H314	Causes severe skin burns and eye damage.
H315	Causes skin irritation.
H317	May cause an allergic skin reaction.
H319	Causes serious eye irritation.
H335	May cause respiratory irritation.
H336	May cause drowsiness or dizziness.
H400	Very toxic to aquatic life.
H410	Very toxic to aquatic life with long lasting effects.
H411	Toxic to aquatic life with long lasting effects.
R10	Flammable
R22	Harmful if swallowed

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UMONIUM38 INSTRUMENTS

Safety Data Sheet

according to Regulation (EC) No. 1907/2006 (REACH) with its amendment Regulation (EU) 2015/830

R34	Causes burns
R38	Irritating to skin
R43	May cause sensitisation by skin contact
R50	Very toxic to aquatic organisms
R50/53	Very toxic to aquatic organisms, may cause long-term adverse effects in the aquatic environment
C	Corrosive
N	Dangerous for the environment
Xi	Irritant
Xn	Harmful

SDS EU (REACH Annex II)

This information is based on our current knowledge and is intended to describe the product for the purposes of health, safety and environmental requirements only. It should not therefore be construed as guaranteeing any specific property of the product

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TCB
INSURANCE CONSULTANTS & BROKERS

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